

## PATTERN FOR FACE MASK – Validated by Grenoble Hospital (FR)

### I. Aim

To overcome the shortage of medical face masks, the University Hospital of Grenoble (France) revisited the idea of the face mask published on [carftpassion.com](http://carftpassion.com) in order to create a prototype usable for all non-medical staff. The end goal of this measure is to keep the stock of medical face mask for the Health Care System as we face the Covid-19 crisis. Let's all use it to protect ourselves and others (*#Solidarity #Eco-friendly #Health&Safety*).



### II. Mask creation & maintenance

#### 1. Materials:

- Sewing essential
- 2 pieces of prewashed cotton fabric (12" x 6") for the outside and the lining
- 1 thin piece of prewashed fleece fabric (12" x 6")
- Round Cord Elastic Band (1/16")



This reusable face mask is made of 2 layers of cotton fabric (top + bottom) and 1 fleece fabric layer placed in between.

#### 2. Maintenance:


Wash the face mask daily at 30°C with a conventional detergent.

#### 3. Creation:

|  |
|--|
| 1- Cut out the pattern of your size  |
| 2- <a href="#">#PiecesA</a> : using this pattern, cut out 4 pieces of cotton fabric (2 for the outside layer + 2 for the lining layer)                               |
| 3- <a href="#">#PiecesB</a> : using the same pattern, cut out 2 pieces of fleece fabric  |
| 4- Measure & cut the length of the elastic band in two pieces of approximately 6"-8"   |
| 5- <a href="#">#OutsideLayer</a> : turn two <a href="#">PiecesA</a> with the right side facing each other on the centreline & sew a straight line down               |
| 6- <a href="#">#LiningLayer</a> : repeat the previous step using the remaining two <a href="#">PiecesA</a>   |
| 7- <a href="#">#FleeceLayer</a> : repeat step 5 using the two <a href="#">PiecesB</a>  |
| 8- Align <a href="#">OutsideLayer</a> with <a href="#">LiningLayer</a> and insert the <a href="#">FleeceLayer</a> in between. Sew the top & bottom lines of the mask |
| 9- Turn the mask upside down (right side out) and topstitch the centreline, the top & the bottom of the face mask  |
| 10- Fold in 1/4" the two raw edges of the side ear of the mask and sew a straight line down. Topstitch if necessary  |
| 11- Insert the elastic band through the pocket and tie a dead knot to the ends. Repeat the same to the other side  |



# Face Mask Pattern

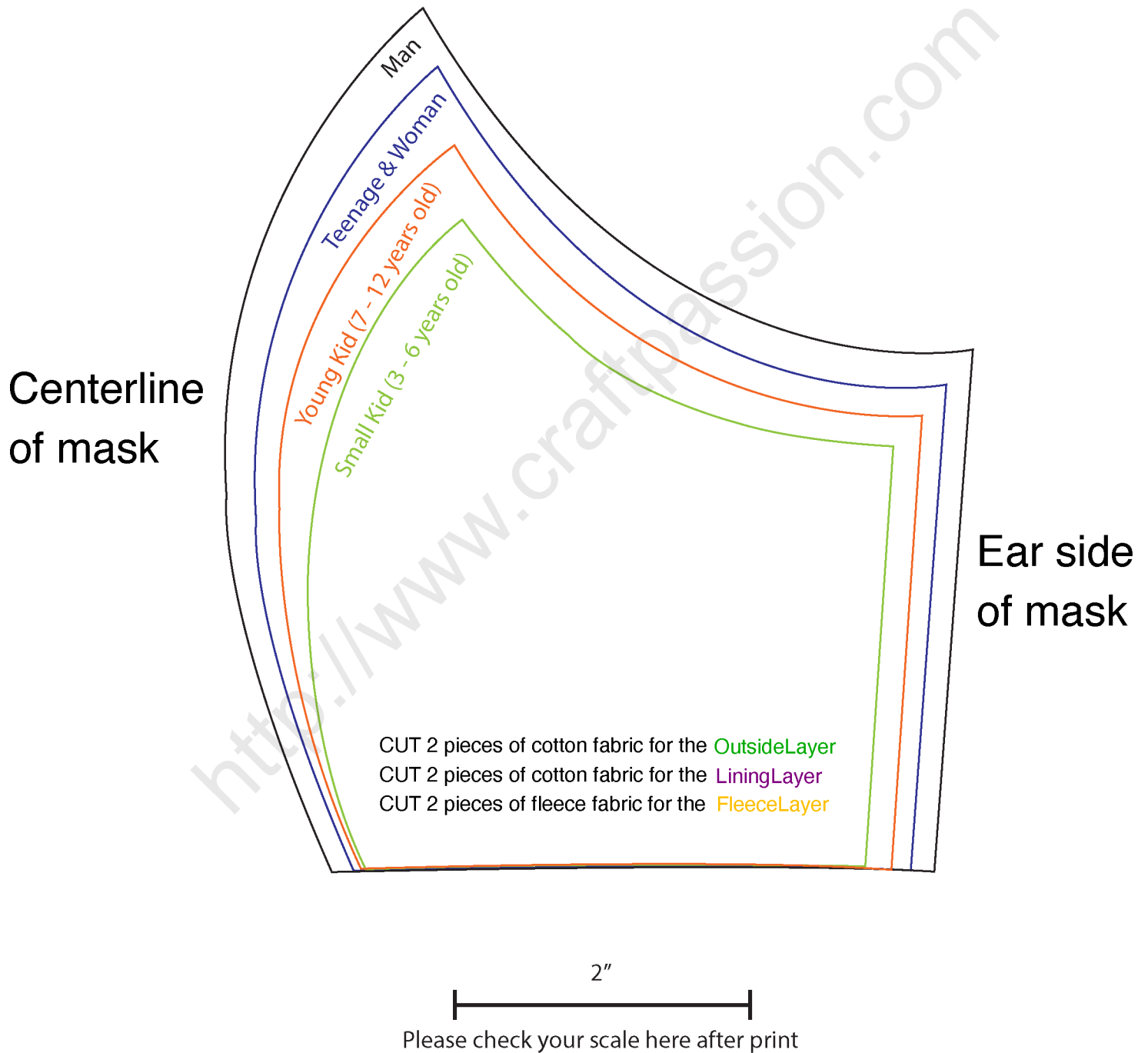

Note: Please add 1/4" seam allowance for all the pattern